## **TRIX**

### **Gebrauchsanleitung** **User's Manual**

**Komponenten:**

**Components:**

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- 1. Main frame
- 2. Quick-adjust assembly
- 3. Front boom
- 4. Internal gear hub
- 5. Handlebar
- 6. Seat
- 7. Twist shifter
- 8. Brake lever
- 9. Seat pouch
- 10. Seat adjustment
- 11. Serial number/  
Bike coding
- 12. Track adjustment
- 13. Pedals with Toe clips and  
straps

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## Contents

<b>Options and Accessoires</b> .....	<b>2</b>
<b>Description</b> .....	<b>3</b>
What is the Trix .....	3
Intended use.....	3
Indications for use.....	3
Contraindications.....	3
Adaptability to individual needs .....	3
Multiple users .....	3
Protection against environmental influences.....	3
Cleaning and disinfection .....	3
Applicable standards and regulations.....	3
<b>Safety information</b> .....	<b>4</b>
Read User's Manual.....	4
Before your first ride.....	4
Observing traffic regulations .....	4
Avoiding muscle strain.....	4
Tipping and overturning .....	4
Braking .....	4
Suitable clothing and helmet .....	5
Risk of injury from rotating parts.....	5
Crushing hazard.....	5
Risk of flammability .....	5
Pedals.....	5
Use of a seat pad.....	5
Being visible for others.....	5
Riding at night.....	5
Technical condition .....	5
Replacement parts and accessories .....	5
Pulling a bicycle trailer .....	6
Loading the Trix .....	6
Serial number and bike coding.....	6
<b>Adjusting your Trix</b> .....	<b>7</b>
Checking the frame length.....	7
Adjusting the frame length .....	7
Checking the tracking adjustment .....	9
Adjusting the steering.....	9
Adjusting the handlebar width/angle .....	10
Adjusting the seat back.....	10
<b>Riding your Trix</b> .....	<b>11</b>
Attaching the flag .....	11
Get in and out.....	11
Correct tire pressure.....	11
Shifting.....	12
Shifting gears with the twist shifter .....	12
Shifting gears with the bar end shifter (optional).....	12
Braking .....	12
Transporting luggage.....	13
Seat pouch and fender stay pouches.....	13
Pulling a trailer or a second Trix in tandem .....	13
Parking brake .....	14
Parking the Trix upright .....	14

Trix with differential.....	14
Adjusting the suspension.....	14
Transporting the Trix.....	15
Packaging the Trix.....	16
Transporting the Trix on a car rack.....	16
Removing and fitting the front boom .....	16
Checking and adjusting the alignment of the front boom .....	17
<b>Maintenance and repair</b> .....	<b>18</b>
Maintenance intervals .....	18
Checking wheels for lateral/radial true .....	19
Checking tires.....	19
Changing tires or repairing a flat.....	19
Adjusting the fenders .....	20
Checking the disc brakes.....	20
Checking and adjusting the internal gear.....	20
Checking the chain.....	21
Cleaning and oiling the chain.....	21
Checking the chain tube .....	22
Checking the frame quick adjust assembly .....	22
Cleaning and disinfecting your Trix .....	22
<b>Appendix for dealers</b> .....	<b>23</b>
Technical data.....	23
Tightening torques.....	24
Accepting delivery .....	24
Unpacking and checking contents.....	24
Assembling the Trix (USA shipment).....	24
Assembling the Trix (EU shipment).....	25
Adjusting the tracking.....	26
Removing the wheels .....	28
Installing the wheels .....	28
Removing the differential and replacing the sproket.....	29
Adjusting the frame quick adjust assembly .....	31
Replacing the handlebar bearing.....	33
<b>Replacement parts</b> .....	<b>34</b>
Trix complete.....	34
Frame clamp.....	35
Frame quick adjust assembly.....	36
Assembly parts for Shimano Nexus internal gear hub.....	37
20" Fork with headset.....	38
16" Fork with headset.....	39
Drive assembly .....	40
Drive assembly: non-driven wheel .....	41
Drive assembly: differential.....	42
<b>Conditions of warranty and liability</b> .....	<b>43</b>
<b>Inspection pass</b> .....	<b>44</b>

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Pedalpendel  
Pedal Pendulum

Spezialpedale  
Special Pedals

LED-Blinkanlage  
LED blinker system

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## Description

### What is the Trix?

The Trix is a recumbent children's trike and an approved medical device as per the European Directive <sup>93/42</sup>EEC (Medical Device Directive). The trike is driven by leg power via the pedals, which are specially designed to prevent the feet from slipping out when riding. The transmission is a fully enclosed 8-speed hub gear system, which drives one or, optionally, both rear wheels. The padded seat has been specially designed and constructed with flexible materials to cushion the ride and offer maximum comfort. The Trix has a reliable brake system with two independently operated disc brakes.

### Intended use

The Trix is intended for use on paved roads and paths. It can also be adapted for adult use. The required technical adjustments, however, must be performed by an authorized retailer.

The Trix is not approved for off-road use or racing. It should not be ridden in traffic until it has been equipped with the accessories required by the applicable traffic regulations of the country in which it is being used.

### Indications for use

The Trix is designed for children with neuromuscular disorders, such as cerebral palsy or muscular dystrophy, who – owing to special physical conditions – are unable to ride standard children's bicycles or vehicles, even with training wheels. Other indications include coordination disorders, balance disorders and restricted movement associated with spasticity, paraplegia, dysmelia, amputations, etc.

### Contraindications

The Trix is not suitable for children with visual impairment.

### Adaptability to individual needs

The Trix can be equipped to accommodate riders with various physical impairments. The following are just a few examples of available accessories: single-hand controls, crank shorteners, special pedals with or without calf support, pedal pendulums, lap and shoulder belts, and holders for walking sticks.

### Multiple users

Trix trikes that have been used in accordance with their intended use can be re-used by a new rider after the trike has undergone an inspection by an authorized retailer.

### Protection against environmental influences

The Trix is constructed using special materials that protect it against corrosion: powder-coated aluminum frame parts, plastics, and stainless steel.

### Cleaning and disinfection

The handlebar grips and seat can be cleaned and disinfected using household cleaning agents and disinfectants. For more information, see the section "Cleaning and disinfecting your Trix" on page <sup>26</sup>.

### Applicable standards and regulations

The Trix has been designed and manufactured in compliance with the following regulations and standards: European Directive <sup>93/42</sup>EEC (Medical Device Directive)

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## Safety information

### Read User's Manual

Read this User's Manual carefully and take the time to familiarize yourself with the construction and function of the Trix in practice. Share this information with the child who will be riding the Trix. Be sure that the child has understood all safety-related functions.

### Steps Option

The frame of the Trix has been tested in combination with the optionally available Shimano Steps pedal-assist system and satisfies the requirements of the safety standard EN 15194 for EPACs. All Hase Bikes purchased with a Shimano Steps pedal-assist system come with user manuals from the manufacturer Shimano, which include all the information you need for the safe use of the Shimano Steps pedal-assist system. Upon receipt of your Hase Bikes vehicle, please check to make sure that the current Shimano user manuals on the Shimano Steps E6000 series and the Shimano eTube project are included in the scope of delivery.

### Before your first ride...

Your authorized retailer has assembled your new Trix and adjusted it to your size requirements. Please refer to the section "Adjusting your Trix" on page 9 if you need to make adjustments for another rider. Before setting out on your first trip, please take some time to familiarize yourself with the Trix. Due to its unique construction, there are significant differences in handling in comparison with a standard bicycle. Accompany the child on a test ride in an open area away from traffic, and allow the child to practice riding until s/he has mastered all important riding skills.

### Observing traffic regulations

The Trix comes with a lighting system and all equipment required by the German road traffic regulations (StVZO). Any changes, adjustments, or repairs must be made by a qualified bike mechanic. Traffic regulations are different in every country. It is important to ensure that the Trix is properly equipped for traffic in the respective country prior to use.

In Germany, children under the age of 8 must ride on sidewalks accompanied by an adult, even if the respective road has a bicycle lane. Children who are at least 10 years of age can ride in traffic accompanied by an adult. Between the ages of eight and ten, children can ride on sidewalks or in bicycle lanes. Be sure to follow the applicable regulations in the country of use, and observe the respective rules for adult supervision.

### Avoiding muscle strain

In a recumbent position, the rider uses different muscle groups than with a standard bicycle. Riders with little or no experience on recumbent bikes should begin with short, easy trips in order to avoid excessive muscle strain.

### Tipping and overturning

Recumbent tricycles have a low center of gravity. This allows for cornering at higher speeds. The centrifugal force felt when turning at high speeds, however, can cause the trike to skid or even tip over, which could cause injury to the rider or others. Avoid making hectic steering movements or braking in curves. This increases the trike's tendency to tip.

Be careful when stopping on inclines. The Trix could roll backwards and, if brakes are applied abruptly, even overturn. Always wear a helmet for your own safety.

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## Braking

The disc brakes of the Trix are very powerful and have good modulation. However, it is still possible for the wheels to become locked. Before your first trip in public traffic, be sure to take the time to familiarize yourself with the braking strength by trying a few braking trials in an open area away from road vehicles.

## Suitable clothing and helmet

Do not wear loose clothing that could get caught in the wheels or gears. Beware of scarves and clothing strings/cords that hang down. Always wear a helmet in order to avoid head injury in the case of an accident.

## Risk of injury from rotating parts

If a patient seated in a trike that is being pulled as a trailer allows his/her hands or fingers to hang down beside or below the seat, there is a risk that they could get caught in the disc brakes and/or wheels. To prevent the associated injuries, it is essential to ensure that patients keep their hands/fingers on the handlebar grips or above the seat at all times!

## Crushing hazard

The Trix has been designed with numerous adjustment sites to make it adaptable to many different riders. Therefore, it was not possible to completely avoid joints that pose a crushing hazard. To prevent injury, be careful not to get your hands or feet caught or pinched in the trike's parts when riding or adjusting the Trix. Be sure to discuss these dangers with the Trix rider.

## Risk of flammability

The upholstery material of the seat cover is not resistant to ignition sources as specified in EN 1021-2. Therefore, be sure to keep the seat away from open flames, heat sources, etc.

## Pedals

In a recumbent position, it is especially easy for the rider's feet to slip from the pedals. Therefore, be sure to use the mounted pedals with toe clips and to secure the feet using the attached straps to prevent them from slipping out of the toe clips.

## Use of a seat pad

Please keep in mind that using a seat pad to raise the sitting position can change the center of gravity and thus also the handling of the Trix. Therefore, before riding in traffic, the rider should take time to familiarize herself with the handling of the Trix with the seat pad in an open area away from traffic.

## Being visible for others

The Trix is lower than an upright bicycle. This may make it more difficult for other road vehicles to see you. Always ride with the flag supplied with your Trix. It attracts the attention of other road users.

## Riding at night

Always use lights when riding at night or in conditions of low visibility, such as dusk and dawn.

## Technical condition

The Trix should only be ridden in perfect technical condition. Check for damage before each trip. Take the Trix to your bike shop for regular maintenance. The recommended maintenance intervals can be found in the section "Maintenance intervals" on page 21.

## Replacement parts and accessories

Use only the replacement parts and accessories recommended by Hase Bikes for the Trix. If other parts are used, the warranty will lose its validity. Hase Bikes is not liable for any resulting damage. Consult your bike shop for more information.

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**Pulling a bicycle trailer**

Accessories are available for attaching bike trailers to the Trix. If you live in Germany, however, keep in mind that according to German road traffic regulations (StVZO), only riders who are at least 16 years of age are allowed to pull a bike trailer.

**Loading the Trix**

Be sure not to exceed the maximum permissible combined weight for rider and luggage: 140 kg (308 pounds). All luggage should be secured using the mount behind the seat or a lowrider rack (accessory). Always ensure that the load is distributed evenly.

**Serial Number and Bike Coding**

The serial number can be found on the end of the main frame (see illustration on the inside cover of this manual). This is the only place where bike coding can be engraved. Bike coding helps authorities locate the bicycle owner in the case of theft or loss.

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## Adjusting your Trix

The proper adjustment of the Trix is important for riding comfort and performance. The Trix can be adjusted to the dimensions of the riders from approx. 1.20 to 1.65 m (4' to 5'5") in height or up to 2 m (6'7") with change of crank. Models with a quick-adjust frame can be adjusted to any length within the maximum range of 25 cm. Larger adjustments require a modification of the quick-adjust assembly and chain length.

### Checking the seat position

Have the child sit in the seat of the Trix and place her feet on the pedals with the balls of her feet on the pedal axles. When the crank arms are lined up with the front boom, one leg should be almost straight (see above photo) and in a comfortable position. If the leg cannot be straightened or the foot cannot fully reach the pedal, then the length must be readjusted.

### Adjusting the frame length

1. Loosen the two Allen bolts between the front boom and the main frame <sup>(1)</sup> using a 6 mm Allen key. Position yourself on the right side of the Trix (when facing in the direction of travel).
2. Grasp the front wheel in one hand and the handlebar in the other.
3. Pull the wheel and the handlebar toward one another in order to decrease the length or push them away from one another to increase the length.

#### Note:

If you want to adjust the length by more than approx. 15 to 20 cm (6 to 8 in), you may have to adjust the length of the steering rod as an intermediate step (see Adjusting the steering, page 8) and then continue with the frame length adjustment.

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**Caution!**

The front boom cannot be extended past the minimum insertion mark (“STOP”), which is located on the top of the boom. This mark should not be visible. If the “STOP” mark is visible, the front boom has been extended past its minimum insertion length.

Riding with an over-extended boom could cause accidents or damage to the frame.

**Note:**

For models without a quick-adjust frame (which are sold pre-adjusted to the rider’s height), the chain must be shortened or lengthened whenever the frame length is adjusted.

4. Check the frame length adjustment for the rider (as described above). If further adjustment is required, repeat steps 2 to 4.
5. The bottom bracket shaft must be horizontal. See page 16.
6. Tighten the two Allen bolts (1) between the front boom and the main frame using a 6 mm Allen key and a torque of 23 Nm (17 ft-lbs).

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7. Adjust the steering. See page 9.

**Note:**

Longer crank arms are also available (accessories) for taller riders. These allow the Trix to grow with the child into adulthood. When converting the Trix for adult use, the chain may also have to be lengthened.

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### Checking the tracking adjustment

The rear wheels have a slight camber (ie: tilt inward) for increased cornering stability. Because the frame does not lie parallel to the road surface, changes in tracking can result from adjustments in the length of the frame. The wheels may no longer be parallel to the direction of travel so that major friction is produced between the wheels and the road when riding. This increases road resistance and tire wear.

Trix models without quick-adjust frames are factory-adjusted to the specified height of the rider. Trix models with quick-adjust frames are pre-set to a length that accommodates riders of 175 cm (5'9") in height. The tracking must only be adjusted if the frame length is shortened or extended by more than 20 cm (8 in), if the front boom is replaced with a lowered boom, if a suspension fork is installed, or if the front wheel is replaced by a larger or smaller wheel. Tracking adjustments should only be made by a qualified bicycle mechanic.

### Adjusting the steering

The movements of the handlebars are conveyed to the front wheel by means of a steering rod <sup>(1)</sup>. The steering rod consists of two tubes, the smaller of which fits inside of the larger and can be moved to adjust the length. Their position is fixed by means of a clamping bolt. The steering rod must be readjusted if the length of the Trix has been changed.

1. Loosen the Allen bolt <sup>(2)</sup> on the steering rod (left side of the Trix) with a 5mm Allen key. Position the front wheel so that it is pointing forward and hold it in this position.
2. Position the handlebar so that it is perpendicular to the main frame.
3. Tighten the Allen bolt <sup>(2)</sup> on the steering rod with a 5mm Allen key.
4. Check that the handlebar has full freedom of movement by moving it as far as it will go to the left and to the right.

#### Caution!

Any restrictions in handlebar movement could lead to accidents with possible injuries.

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### Adjusting the handlebar width / angle

You can adjust the width and angle of the handlebar to fit your shoulder width and arm length

1. Loosen the bolts <sup>(1)</sup> on each side of the handlebar using a 6mm Allen key
2. Pull out the handlebar grips in order to increase the width or push the grips further in to decrease the width

**Caution!**

The lower ends of the grips should not be visible through the slots in the center section of the handlebar. This means they have been pulled out past their minimum insertion length. Major risk of accident! If too much pressure is placed on the grips, the center section could break.

Make sure that the handlebar grips are not pulled out so far that they bump against the wheels or fenders. Major risk of accident! The steering could lock in the handlebar's maximum left or right position.

### Adjusting the seatback

1. Release the locking mechanism of the seat by pressing down on the cable (shown yellow in diagram)
2. Loosen the two quick releases <sup>(2)</sup> of the seat stays
3. Slide the quick releases into the desired position
4. Re-tighten the two quick releases <sup>(2)</sup>
5. Press gently on the seat to lock it back into place on the seat stay

**Note:**

You can also use the upper slots for the quick releases in order to recline the seatback even further.

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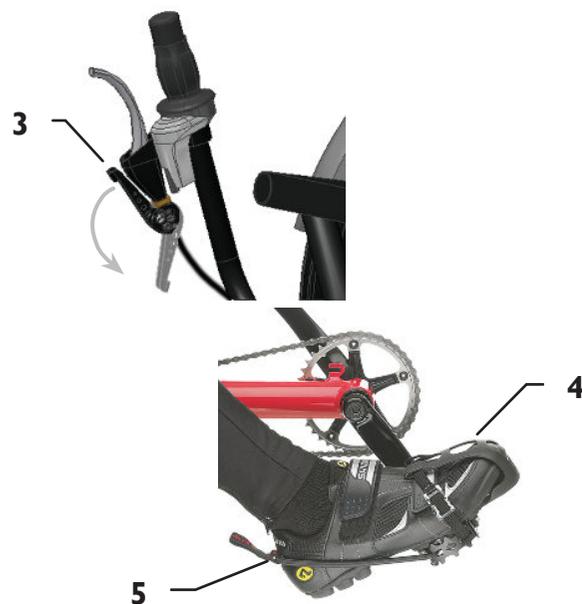
### Attaching the flag

#### Caution!

The Trix is lower than an upright bicycle. This may make it more difficult for other road vehicles to see you. Use the included flag at all times.

Push the pole of the Trix's safety flag through the loop <sup>(1)</sup> on the left side of the seat cover (or on the right side for driving on the left) and brace the end by wedging it between the tubes of the seat stay <sup>(2)</sup>.

### Correct tire pressure and technical condition

The Trix should only be ridden with the correct tire pressure. The tire pressure range is specified on the tire sidewalls. The two rear wheels are equipped with spoke guards. Therefore, in order to inflate the tires, you must first pull apart the Velcro opening. The Trix should not be ridden unless it is in perfect working order.

#### Caution!

Tire pressure affects the handling and safety of the Trix. Therefore, always ride with the tire pressure indicated on the side of the tire.

### Get in and out

The parking brake prevents the Trix from rolling when mounting or dismounting:

1. Engage the parking brake by pulling down the parking brake lever <sup>(3)</sup> on the right handlebar grip. For hydraulic brakes (optional) compress the right brake lever and wrap the Velcro strap around the brake lever to hold it in an engaged position.
2. The parking brake is now engaged.
3. Sit down on the seat.
4. Place one foot on the respective pedal, sliding it into the toe clip <sup>(4)</sup> and pull the elastic strap <sup>(5)</sup> around the heel.
5. Place the other foot on the other pedal, sliding it into the toe clip and pull the elastic strap around the heel.
6. Fasten the lap and shoulder belts (accessories) if available.
7. Disengage the parking brake by moving the lever back to its original position or – for hydraulic brakes – removing the Velcro strap.

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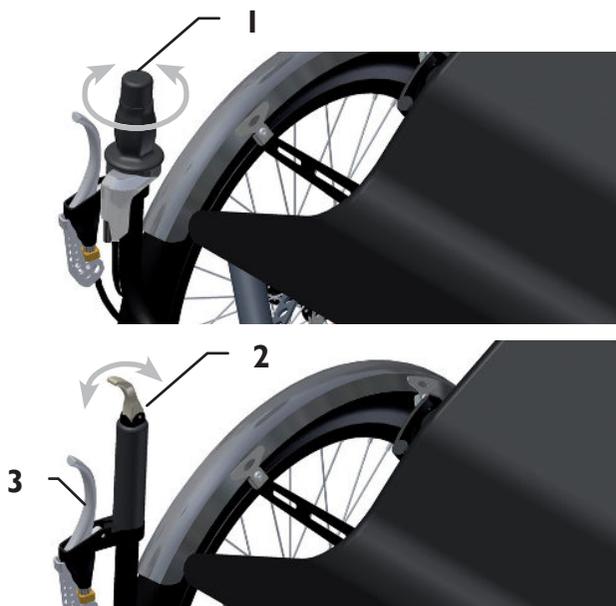
## Shifting

Shifting gears allows you to adjust pedaling strength to the terrain and desired speed. The force required per pedal revolution can be decreased by selecting the appropriate gear. In this way, it is much easier to ride up a steep hill in a lower gear than in a high gear. In a lower gear, however, you are slower and have to pedal faster. High gears are best suited for riding at higher speeds on flat terrain or riding downhill. Select a gear that allows you to pedal at more than 70 pedal revolutions per minute. This cadence is optimal for producing sustained power and preventing excessive strain on the knee joints.

Trix is equipped with an 8-speed hub gear with twist shifter. Hub gears allow you to shift even when the pedals are not in motion.

### Caution!

Please note the information provided in the enclosed manual from the gear system manufacturer.

### Caution!

Before riding the Trix on public roads, be sure to familiarize yourself with gear shifting in an open area away from traffic. Shifting gears takes practice, which could distract you from navigating road traffic.

## Shifting gears with the twist shifter

Turn the twist shifter <sup>(1)</sup> on the right handlebar clockwise to shift to a higher gear for faster riding or counterclockwise to shift to a lower gear for slower riding.

## Shifting gears with the bar end shifter (optional)

Move the lever at the tip of the right handlebar grip <sup>(2)</sup> forward, away from you, to shift to a higher gear for faster riding or backward, toward you, to shift to a lower gear for slower riding.

## Braking

In its standard version, the Trix is equipped with mechanical disc brakes. Hydraulic disc brakes are also available. The brake levers <sup>(3)</sup> are mounted on the left and right grips of the handlebar. The left wheel brake is controlled with the left brake lever and the right wheel brake with the right brake lever.

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**Caution!**

The disc brakes of the Trix are very powerful and have good modulation. However, it is still possible for the wheels to become locked. Before riding the Trix on public roads, be sure to familiarize yourself with the braking force by trying a few braking trials in an open area away from traffic.

**Transporting luggage**

Special mounts <sup>(1)</sup> for a bicycle bag are located behind the seat. These mounts are suitable for various models of bags. We recommend the appropriate Ortlieb bag for recumbents. Other hook-mounted bicycle bags, such as VauDe bags, can also be used. However, please make sure that these bags cannot come in contact with the rear sprocket.

**Note:**

When transporting larger quantities of luggage, be sure to distribute the load evenly. An unbalanced load can cause difficulties in road handling.

**Seat pouch and fender-stay pouches**

A small zippered pouch <sup>(2)</sup> is built in to the seat back. This pouch can be used for storing small objects. One or two additional pouches (optional) can be mounted on the fender stays <sup>(3)</sup> for transporting small articles such as keys, sunglasses, etc.

**Note:**

These pouches are also perfect for keeping our Hase Bikes postcards on hand for interested passers-by. This saves you from having to make lengthy explanations. The postcards can be reordered from our office at any time.

**Pulling a trailer or a second Trix in tandem**

With the universal coupling <sup>(4)</sup> (accessory), the Trix can be used in combination with a second Trix or Kettwiesel as a tandem trike. You can also attach a trailer with a Weber coupling, a BOB trailer, or a Weber Monoporter. If you live in Germany, however, keep in mind that according to German road traffic regulations (StVZO), only riders who are at least 16 years of age are allowed to pull a bike trailer. The Trix can also be pulled as a trailer behind a Kettwiesel.

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### Parking brake

The Trix is equipped with a parking brake to prevent it from rolling when parked. Trix with mechanical disc brakes have a lever <sup>(1)</sup> on the right handlebar grip.

1. Move the lever forward.
2. The brake is now engaged.
3. Move the lever back. The brake is disengaged.

For Trix with hydraulic disc brakes, a Velcro strap <sup>(2)</sup> is attached to the left handlebar grip.

1. Pull the brake lever.
2. Wrap the Velcro strap <sup>(2)</sup> around the brake lever and attach the strap to the grip. The brake is now engaged.
3. Remove the Velcro strap <sup>(2)</sup>. The brake is disengaged.

### Parking the Trix upright

The Trix can be parked upright with the front wheel raised. In this position, it requires less space for parking than a standard bicycle. A plastic parking stand is mounted behind the seat to protect the Trix from damage in this position. To bring the Trix into the upright parking position, simply lift the front wheel.

### Trix with differential

The optional differential offers even more safety and stability on loose road surfaces. It transfers the power to both rear wheels.

#### Note:

Due to the construction of the differential, there is some bearing play in the left axle when not under load (i.e. when no force is applied to the pedals). However, there is no play in this bearing when the rider is pedaling.

### Adjusting the suspension

For trikes with full suspension, the rear suspension system can be adjusted to the total weight and the rider's handling preferences. Pressurize the two rear shocks <sup>(3)</sup> to the desired air pressure using a shock pump. Be sure to fill both shocks to the same pressure. The reference values shown on the shocks can be helpful in deciding what pressure setting to use.

#### Caution!

Never pressurize the shocks to more than the maximum pressure of 19 bar (130 psi). Excessive air pressure can damage the shocks. Always wear safety goggles for eye protection when working on the suspension system.

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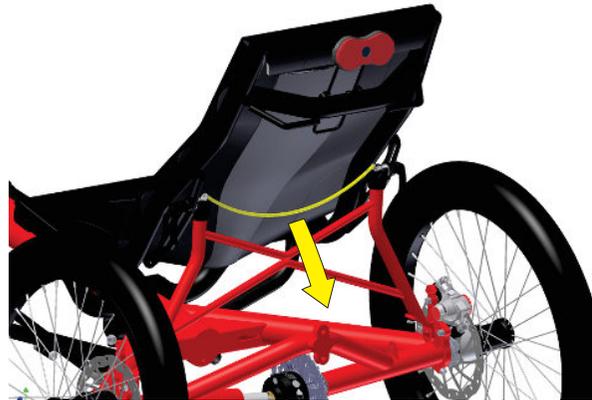
### Transporting the Trix

The Trix can be adjusted to its most compact state for transportation by car, train, or other vehicle (see also page 6, Adjusting the frame length). The front boom can be fully inserted or removed, and the seat can be folded forward or backward.

1. Before shortening the frame for transport, make a note of the position of the front boom in the main frame (using the length-adjustment markings). This will make it easier to adjust the Trix to the required length after transport.
2. Adjust the frame to its minimal length. For more information, see section “Adjusting the frame length” on page 6.
3. Loosen the quick release on the fork and remove the front wheel.

**Caution!**

Be careful not to damage the front fender.

4. Release the locking mechanism of the seat by pressing down on the cable (shown yellow in diagram).
5. Fold the seat stay forward.
6. Fold the seat to the back (see middle diagram) or
7. Unfasten the seat cover beneath the seat and fold the seat forward (see lower diagram).
8. Now the Trix is ready for transport.

**Caution!**

Be sure to protect all sensitive parts, such as the disc brakes and rear derailleur, from damage.

If you are planning to take the Trix on an airplane or have it shipped by a forwarding company, we recommend packaging it in a cardboard box for protection.

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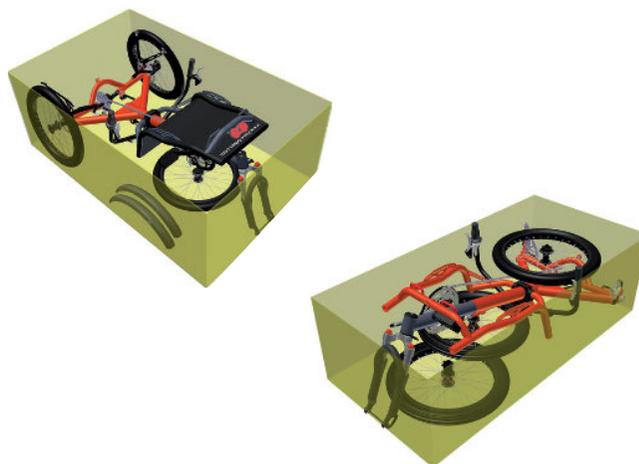
**Packaging the Trix**

Use a cardboard box with internal measurements of at least 162 x 88 x 54 cm (64 x 35 x 22 in) and for the full suspension Trix 176 x 88 x 54 cm (69 x 35 x 22 in). Adjust the frame to its minimal length and remove the front wheel. Place the Trix in the box and secure it with packing material to prevent movement. Be sure to protect all sensitive parts from damage. The packing dimensions can be further reduced by removing the two rear wheels and fenders. See page 28.

**Transporting the Trix on a car rack**

The Trix can be transported with a standard roof or rear-mounted bicycle carrier. The rack must be equipped for three bicycles and adjusted so that each wheel can be anchored on a separate wheel tray. Secure all wheels and the frame using load straps. Check the mounting to be sure that the Trix cannot come loose during the trip. Remove all parts that could fall off in transport.

**Removing and fitting the front boom**

The front boom can be removed from the main frame to further reduce the frame length.

1. Remove the chain from the chainring.
2. Loosen the two Allen bolts (1) between the front boom and the main frame using a 6mm Allen key.
3. Loosen the Allen bolt (2) on the steering rod (3) (left side of the Trix) with a 5mm Allen key.
4. Loosen the Allen bolt (4) on quick-adjust assembly; then pull the front boom out of the main frame while, at the same time, pulling the inner tube of the steering rod assembly out of the outer tube.
5. Be sure to protect the front boom and steering rod assembly from damage.
6. To re-insert the front boom, follow the above instructions in reverse order. Insert the front boom into the main frame, ensuring that the groove plate (see diagram on next page) fits inside the groove in the front boom.
7. Align the front boom (see following section); then adjust the steering (see page 8).

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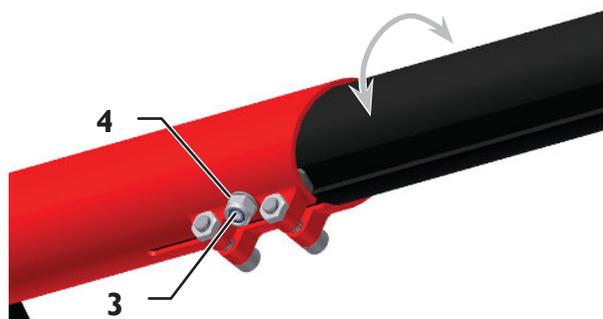
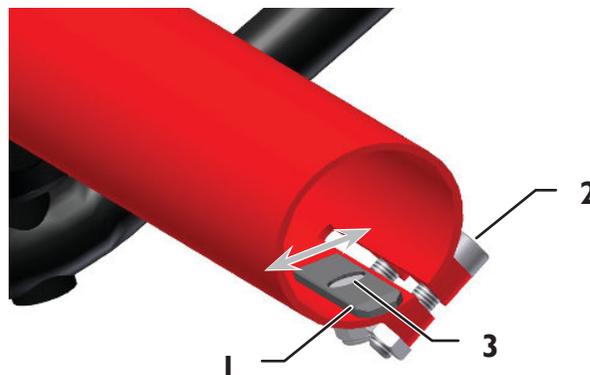
### Checking and adjusting the alignment of the front boom

The front boom must be aligned in the main frame in such a way that the plane of the front wheel is perpendicular (at a 90° angle) to the ground. To ensure that the front boom does not rotate out of alignment (e.g. when adjusting the frame length), it is braced by a small groove plate <sup>(1)</sup> (page 17) inside the frame tube that fits inside a groove in the front boom. This plate has an eccentric screw <sup>(3)</sup> that, when turned, pushes the plate to the side in order to adjust the rotational angle of the front boom. The position of this plate rarely requires readjustment. However, if the front wheel is not positioned perpendicular to the ground, adjust the groove plate as follows:

1. If the front boom has been removed from the main frame (e.g. for transport), insert it back into the frame tube, ensuring that the groove plate <sup>(1)</sup> slides inside the groove of the front boom.
2. Check the alignment of the front boom. It must be positioned in such a way that the plane of the front wheel is perpendicular (at a 90° angle) to the ground.
3. If the two Allen bolts <sup>(2)</sup> between the main frame and the front boom have not been loosened, then loosen them using a 6 mm Allen key.
4. Loosen the locknut <sup>(4)</sup> with a 13 mm wrench and turn the eccentric bolt <sup>(3)</sup> in the groove plate <sup>(1)</sup> using a 4 mm Allen key until the plane of the front wheel is positioned perpendicular to the ground.
5. Then, while bracing the eccentric bolt <sup>(3)</sup> with a 4 mm Allen key, retighten the locknut <sup>(4)</sup> using a 13 mm wrench.
6. Finally, tighten the two Allen bolts <sup>(2)</sup> between the front boom and the main frame using a 6 mm Allen key and a torque of 23 Nm (17 ft-lbs).

**Caution!**

It is absolutely necessary to tighten these bolts with a torque of 23 Nm (17 ft-lbs). Riding with loose bolts poses a high risk of accident and damage to the frame.

7. If applicable, proceed with the remaining steps for fitting the front boom, following the instructions given in the previous section "Removing and fitting the front boom".

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### Maintenance intervals

**Note:**

Please have all inspection and maintenance work performed by your authorized Hase Bikes retailer.

The first inspection should be done after the initial 300 km (200 miles). All additional inspections should be performed in 2000 km intervals or on a yearly basis (see also page 46, "Inspection pass").

Component	Action	Before every ride	Once a month	Once a year
Lighting	Check function	X		
Tires	Check tire pressure	X		
	Check tread wear and sidewalls	X		
Brakes	Check pad thickness		X	
	Check alignment to the brake disc		X	
	Test braking effectiveness	X		
Brake cables or brake hoses	Check for damage	X		
Suspension fork Rear wheel suspension	Check function	X		
		X		
Chain Swing arm chain	Check for wear· oil		X	
	Check for wear· oil			X
Crank bracket	Check bolts			X
Wheels	Check for lateral/radial true		X	
	Check spoke tension		X	
Handlebar	Inspect			X
Steering rod	Check for play			X
Headset	Check for play			X
Hubs	Check for play			X
Pedals	Check for axle play			X
Rear derailleur	Clean· lubricate		X	
Derailleur cables	Check for damage		X	
Quick releases	Check for tightness	X		
Bolts and nuts	Check for tightness		X	
Bottom bracket	Check for play			X
Slot nuts (differential)	Check for damage		X	

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Maximaler Luftdruck  
Maximum Pressure

### Checking wheels for lateral/radial true

The wheels have been carefully trued prior to shipment. During the break-in period, stress on the wheels may cause the spokes to settle into the rims so that the spoke tension is no longer uniform and the wheel untrue. Have your wheels re-trued by your authorized retailer at the first inspection after approx. 300 km (200 miles). Subsequent tensioning is seldom necessary. Check the spoke tension and lateral/radial true of the wheels regularly. You can check the spoke tension by squeezing together two adjacent spokes, working all the way around the rim. If you notice major differences in spoke tension, the wheel should be re-trued. You can check whether a rim is true by attaching a wire to the fork or fender stay and positioning it so that it almost touches the rim. Lift the Trix and rotate the wheel. If portions of the rim move away from or come in contact with the wire, the wheel must be taken to a qualified bicycle mechanic for re-truing.

#### Note:

The rear wheels do not have to be removed for the truing process.

### Checking tires

Check the tires for damage and tread wear. Tires with cuts, cracks, or worn tread should be replaced. Check tire pressure before each ride and always ride with the pressure specified on the tire sidewalls.

#### Caution!

Never ride with excessive tire pressure (higher than specified on the sidewall). The tire could burst. Risk of accident and injury!

### Changing tires or repairing a flat

The front wheel of the Trix is mounted with a quick release. Simply release it to remove the front wheel. You do not have to remove the back wheels for changing a tire or repairing a flat. The tires can be removed and remounted using a standard tire lever with the wheels attached. When removing and remounting tires, take care not to damage the fenders and check that the rim tape is positioned correctly on the rim.

#### Note:

If you replace the original tire with a tire of a different size, it may be necessary to readjust the fenders and the position of the dynamo.

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### Adjusting the fenders

*Trix without rear suspension*

Before you can make adjustments to the fenders, you must first adjust the angle of the seatback in order to gain access to the fender bolts <sup>(1)</sup>.

1. Loosen the two Allen bolts <sup>(1)</sup> using a 5mm Allen key.
2. Slide the fender stay in the slot until it reaches the desired position, ensuring that the fender does not rub against the tire.
3. Tighten the two bolts <sup>(1)</sup> using a 5mm Allen key.

*Trix with rear suspension*

1. Loosen the two Torx bolts <sup>(2)</sup> using a T<sup>25</sup> Torx key. Loosen the two selflocking M6 nuts <sup>(3)</sup> using a 10mm wrench (spanner).
2. Slide the fender stay into the desired position, ensuring that the fender does not rub against the tire.
3. Tighten the two bolts <sup>(2)</sup> and nuts <sup>(3)</sup>.

### Checking the disc brakes

Please note the information provided in the enclosed manual from the brake manufacturer. Have a qualified bicycle mechanic check the pad thickness regularly. If you notice any unusual braking sounds or inadequate braking strength, consult your bicycle mechanic immediately.

### Checking and adjusting the internal gear

Please note the information provided in the enclosed manual from the gear system manufacturer. The gear hub system has been carefully adjusted at the factory. Over time, however, the gear cables tend to stretch, and the gear hub may no longer function with the same precision. In this case, contact your authorized retailer.

**Note:**

A complete readjustment of the gear hub system should be performed by a qualified mechanic. Incorrect adjustments can cause serious mechanical damage. Please note the information provided in the manual from the gear system manufacturer.

**Caution!**

After the gear hub system has been adjusted, it is absolutely critical that you test the gears in an open area away from traffic before taking the Trix onto public roads. Problems may arise that could cause you to lose control of the Trix.

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## Checking the chain

The trike's chain is longer than that of a standard upright bicycle. A long chain is generally subject to less wear. Therefore, it does not have to be changed as often as shorter chains. Chains on models with hub gears need replacing even less frequently. You can check the condition of the chain using a chain wear gauge from a bike shop. We recommend that you have a qualified bicycle mechanic check the condition of the chain and, if necessary, replace the chain, and possibly also the chainring and cassette.

## Cleaning and oiling the chain

In order to ensure that the chain does not wear out before the end of its service life, it must be cleaned and lubricated on a regular basis. It is also important that all intermediate chains (e.g. for hub gears and the rear suspension system) be cleaned and lubricated. The chains of the optional rear suspension system are completely enclosed by chain guards and therefore only require annual maintenance. To access these chains, loosen the three countersunk bolts <sup>(1)</sup> using a T<sup>25</sup> Torx key until you can remove the chain guard.

1. Pour a small amount of suitable chain cleaner onto a clean, lint-free cotton cloth, and use it to wipe down the chain. When doing so, slowly turn the crank backwards (i.e. opposite the drive direction). Repeat this procedure with clean patches of the cotton cloth until the chain is clean. Allow the cleaning agent to evaporate for approx. 1 hour before applying a lubricant.

### Caution!

Never use solvents or pressure cleaners. These could wash the lubricants out of the chain bearings and shorten the life of the chain.

2. Apply chain oil, grease, or wax evenly to the chain links while slowly rotating the crank.

### Note:

Rain can wash off traces of lubricant, releasing it into the environment. We recommend biodegradable lubricants.

3. Continue rotating the crank to distribute the lubricant.
4. Wait several minutes to allow the lubricant to penetrate. Then wipe off all excess lubricant using a clean, dry cloth.

### Caution!

Never allow chain lubricant to come in contact with the brake discs or tires. This could significantly reduce braking performance or, in the case of lubricant on tires, wheel traction.

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### Checking the chain tube

The purpose of the chain tube <sup>(1)</sup> is to protect your clothing from chain lubricant. The mounting of the chain tube is moveable, allowing it to adjust to the chainline when gears are shifted 'derailleur gears'. Over time, friction from the moving chain will cause the tube to wear. Check the ends of the chain tube for any signs of wear. If the ends are completely worn, have your bicycle mechanic replace the chain tube.

#### Note:

In order to replace the chain tube, the chain must be opened at the master link.

### Checking the quick-adjust assembly

The quick-adjust assembly facilitates frame-length adjustment and protects the chain from the elements. Dirt and grime can negatively affect the functioning of the quick-adjust mechanism. It is essential that the quick-adjust assembly be checked on a regular basis to ensure it is functioning properly. If the assembly is not functioning properly, remove the chain from the front chainring, and clean all parts of the assembly. Then apply a small amount of light (low viscosity) lubricant to the sites where the guide tubes make contact with the housing of the idler pulleys <sup>(2)</sup>, and slide the tubes back and forth in order to distribute the lubricant. Then wipe off any excess lubricant, and pull the chain back around the front chainring.

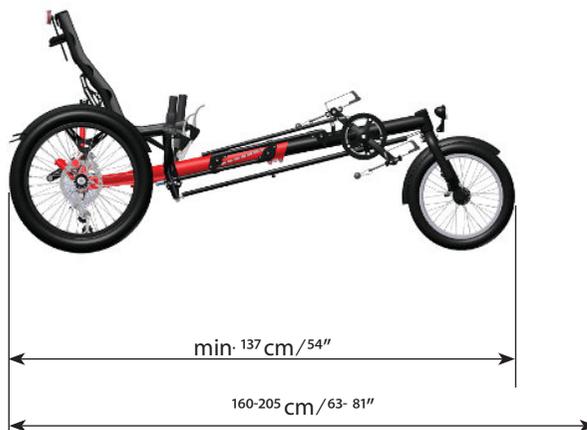
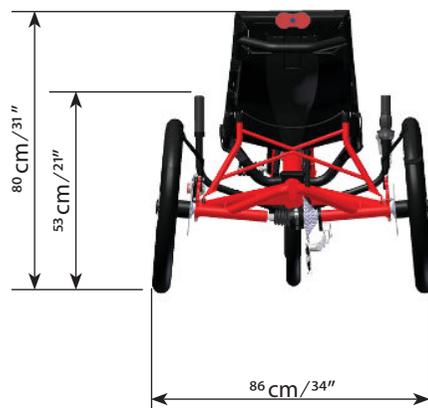
### Cleaning and disinfecting your Trix

Be sure to clean your Trix on a regular basis. Dirt and grime, as well as salt from winter roads, can cause damage. To clean your Trix, use a sponge and soapy water (ideally with a biodegradable household cleaning product). Stubborn marks on metal surfaces with a matt finish can be removed best by using a sponge with a wheel-rim cleaner from an automotive supply shop. You can also clean your Trix with plain water. If you use a hose, be sure to spray the water gently. Never use pressure washers. The high pressure could force water into the bearings, damaging them. Wipe all surfaces with a moist, lint-free cloth. Apply a suitable spray wax or a similar corrosion inhibitor to all surfaces of the Trix except for the brake pads, brake discs, grips, brake/gear levers, chain, seat cover, and tires. Then clean and lubricate the chain. The handlebar grips and seat can be cleaned and/or disinfected using standard cleaning agents and disinfectants. Be sure to use a disinfectant that is suitable for plastic materials.

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## Appendix for dealers

### Technical data

Total length: 160-205 cm (63-81")
Minimum length without front wheel: 137 cm (54")
Total width: 86 cm (34")
Total height: 80 cm (31") (With seat folded: 53 cm / 21")
Bottom bracket height: 47 cm (18.5")
Seat height: 46 cm (18")
Weight: 16.3 kg (35.9 lbs)
Permissible load: max. 140 kg (308 lbs)
Brake system: mech. disc brakes Optional: hydraulic disc brakes
Optional: 8-speed Shimano Nexus
Frame: aluminum powdercoated
Wheels: 20"
Rims: alu double-wall with eyelets
Tires: rear 35-406 (20 x 1.35") to 60-406 (20 x 2.35"), front 50-305 Tire pressure range: use recommended tire pressure shown on sidewalls
Fits riders from 1.20 to 1.65 m (4' to 6'7") in height

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**Tightening torques**

Use a torque wrench for all bolts. The following table lists the standard values for bolts of 8.8 quality (Grade 5)

Bolt	Tightening torque
M 5	4 ft lbs
M 6	6.6 ft lbs
M 8	17 ft lbs
M 10	34 ft lbs
M 12	58 ft lbs

**Accepting delivery**

If you notice the packaging has been damaged, please notify the shipping company directly.

**Unpacking and checking contents**

The Trix is packaged in various states of pre-assembly, depending on the region to which it is being shipped. Open the packaging and remove the shipping supports. Check the contents of the package for agreement with your order.

Additionally, the following items should be included:

Trix User's Manual, as well as manuals from other manufacturers for various components such as gear system, brakes, lighting system, etc., and flag.

**Assembling the Trix (USA shipment)**

1. Insert the front boom into the main frame and tighten the bolts lightly at first.
2. Mount the frame quick-adjust assembly fittings. Put the chain on the sprockets.
3. Mount the steering rod.

**Note:**

Please refer to the instructions for removing the wheels on page 28 of this manual. You can also refer to the diagrams provided in the section "Replacement parts".

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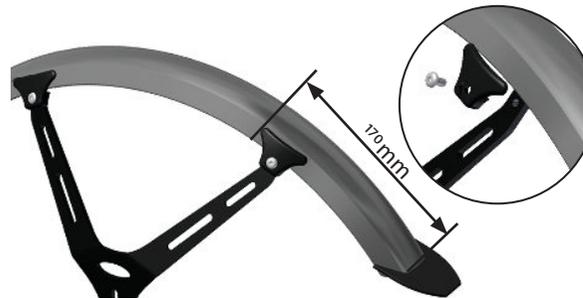
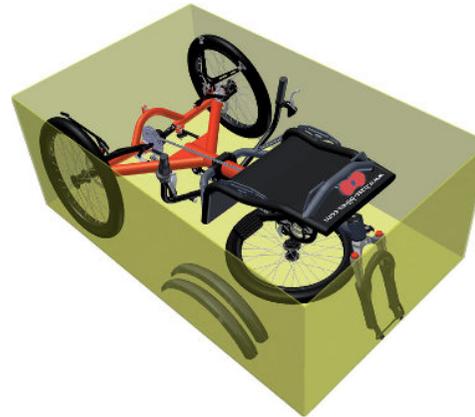
**Note:**

If the brakes begin to drag, they must be realigned.

4. Mount the seat frame to the frame clamps.
5. Mount the fender stays to the swing arms, tightening the bolts by hand.

Proceed with steps listed under "EU shipment".

**Assembling the Trix (EU shipment)**

1. Attach the front wheel.
2. Lift the seat and seat stay. Attach the seat to the seat stay with the quick releases.
3. Mount the seat cover.
4. Attach the fenders to the fender stays.
5. Mount any accessories you may have ordered.
6. Adjust the frame length. See page 6.

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## Adjusting the tracking

The rear wheels have a slight camber (i.e. tilt inward) for increased cornering stability. Because the frame does not lie parallel to the road surface, changes in tracking can result from adjustments in the length of the frame. The wheels may no longer be parallel to the direction of travel so that major friction is produced between the wheels and the road when riding. This increases road resistance and tire wear.

Trix models without quick-adjust frames are factory-adjusted to the specified height of the rider. Trix models with quick-adjust frames are pre-set to a length that accommodates riders of 175 cm (5'9") in height.

The tracking will only require readjustment if the frame length is shortened or extended by more than 20 cm (8 in), if the front boom is replaced with a lowered boom, if a suspension fork is installed, or if the front wheel is replaced by a larger or smaller wheel. For this adjustment, you will need a folding ruler, a pencil, a thin tool (e.g. spoke) and a 17mm wrench (spanner).

1. Place the Trix on a flat surface and brace the tires to prevent rolling.
2. Measure the distance from the center of the wheel bolt to the surface of the ground/floor and make a note of this value.
3. Now use a pencil to mark the measured axle height on the front and rear inner faces of the rims on both wheels.
4. Use the ruler to measure the distance between the rims at the height of the rear markings  $L^1$  (for example, between spoke nipples of the inner rows). Make a note of this value.
5. Repeat the measurement for the front markings  $L^2$  and make a note of the value.
6. Compare the two values. If the values differ by more than 5 mm, then the tracking must be adjusted using the track rods, which pull together or push apart the rear tubes of the frame.

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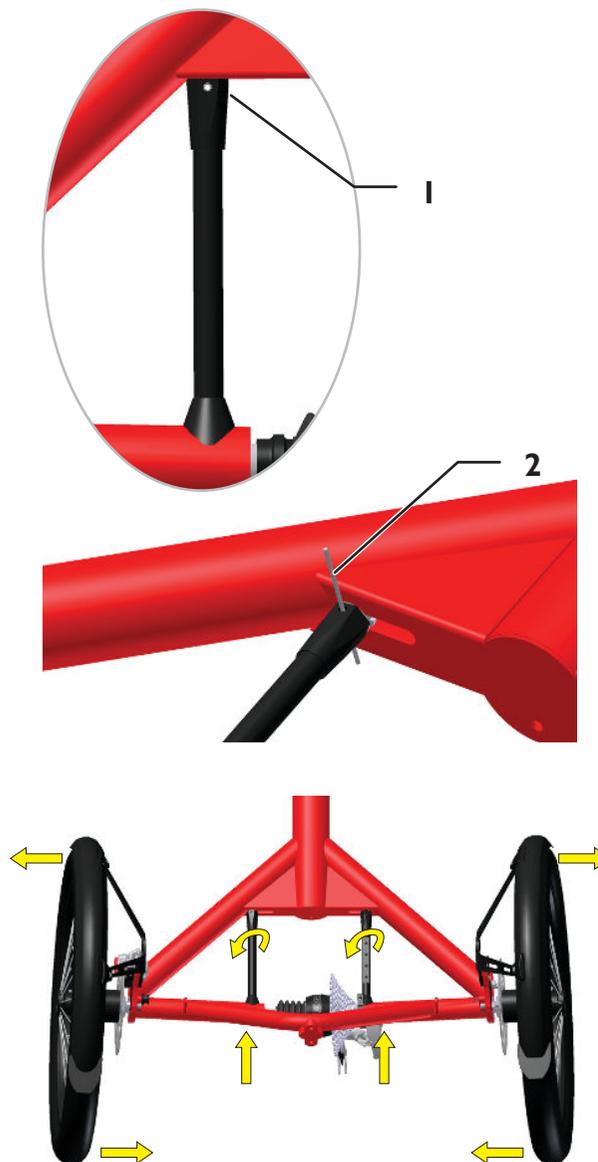
7. Use a 17mm wrench to loosen the jam nut on the left track rod <sup>(1)</sup>, turning it until the hole in the jam nut is lined up with the hole in the track rod.
8. Repeat this procedure with the second track rod.
9. Insert a thin tool, e.g. a spoke <sup>(2)</sup>, through both holes and use it to rotate the bolt inside the tracking rod.
10. If the distance measured between the rear markings of the rims was larger than the distance between the front markings, then rotate the bolts in both track rods counterclockwise by an equal amount (as shown in the diagram below). If the distance measured between the rear markings of the rims was smaller than the distance between the front markings, then rotate the bolts in both track rods clockwise by an equal amount.

**Note:**

Always rotate the bolts in the two track rods by the same amount and in the same direction. For example, if you turn the bolt in the left track rod one full rotation clockwise, then you must also turn the bolt in the right track rod one full rotation clockwise.

**Note:**

One full rotation of the bolt in one track rod is roughly equivalent to a one-millimeter change in tracking.

11. Re-check the distance between the front and rear edges of the wheels by repeating steps <sup>2</sup> through <sup>5</sup>.
12. If the values differ by more than 5 mm, then repeat step <sup>10</sup> until the two values are in agreement.

**Note:**

When you're finished with the tracking adjustment, it is absolutely essential that you remember to tighten the jam nuts on the track rods to prevent the rods from coming out of adjustment.

13. Then tighten the jam nuts against the track rods using a 17mm wrench.

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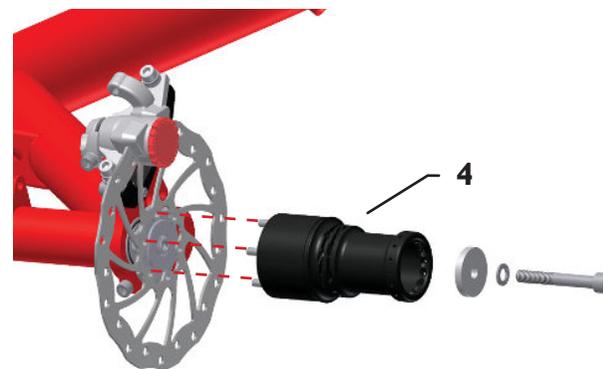
### Removing the wheels

1. Brace the wheel.
2. Unscrew the M<sup>8</sup> allen bolt <sup>(1)</sup> by turning it counterclockwise using a <sup>6</sup>mm Allen key.
3. Hold the brake disc <sup>(2)</sup> and remove the wheel.

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### Installing the wheels

1. Slide the brake disc <sup>(2)</sup> between the two brake calipers from below and hold it against the head of the drive screw <sup>(3)</sup>, aligning the holes.
2. Hold the hub of the left wheel <sup>(4)</sup> against the brake disc <sup>(2)</sup> so that the hub pins protrude through the holes of the brake disc and into the holes in the head of the drive screw <sup>(3)</sup>.
3. Screw the wheel onto the frame with the QS adapter shim, the Nordlock washer and the M<sup>8</sup> allen bolt <sup>(1)</sup> using a <sup>6</sup>mm Allen key, tightening lightly at first.
4. Brace the wheel and tighten the M<sup>8</sup> allen bolt with a torque of <sup>23</sup>Nm (<sup>17</sup>ft-lbs).

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#### Attention!

The M8 allen bolt must be tightened with a torque of 23 Nm (17 ft-lbs). If the bolt is tightened with less torque, the shaft could brake and the drive could fall out.

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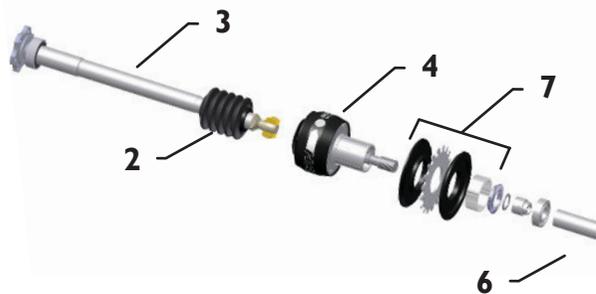
### Removing the differential and replacing the cassette

The differential requires no maintenance and must only be removed when replacing the cassette.

1. Unscrew the left wheel (page 28) or the left brake caliper.
2. Slide the rubber bellows (2) of the differential case (4) to the left.
3. Loosen the M<sup>6</sup> bolt (1).
4. Pull the left axle assembly (3) approx. 10 cm (4") out of the frame tube and out of the differential.
5. Brace the right wheel while unscrewing the differential (4) with the chainring assembly (7) from the right drive shaft (6) clockwise (Attention: left-hand thread) by using a 10mm Allen key (5) in the hexagon socket.

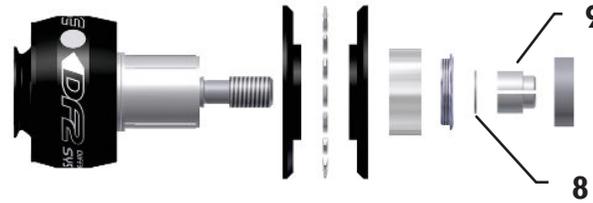
**Note:**

If the right drive screw comes off instead of the differential, then proceed as follows: Loosen the clamp screw on the derailleur hanger and pull the right drive shaft with the differential slightly out to the left. Brace the shaft using a pipe wrench, and unscrew the differential, using a 10mm Allen key (5) in the hexagon socket. When reassembling the unit, the drive screw must be secured using high strength threadlocker and tightened with a torque of 80 Nm (65 ft-lbs).

6. Remove the differential with the chainring assembly. Now the chainring with the chainguards and spacer can be removed.

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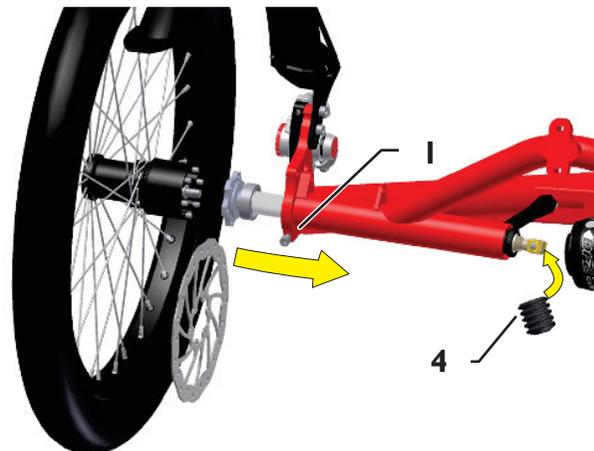
7. Brace the cassette using a chain whip and unscrew the sprocket bolt with a locking removal tool.
8. Pull the cassette off of the freewheel unit.
9. Mount the new cassette. Follow these instructions in the reverse order.

**Note:**

When installing the new cassette, make sure that the shim (8) is lying flat and not wedged at an angle beneath the two-part bearing seat (9).

**Note:**

Lubricate the contact surfaces (10, yellow marked) in the differential with a solid lubricant before you insert the left axle assembly into the hexagon socket.

10. Slide the rubber bellows (4) back onto the end of the left axle assembly. Align the rectangular socket in the side of the differential with the slot nuts on the end of the left axle assembly. Then slide the left axle assembly as far as it will go into the axle tube while guiding the slot nuts into the rectangular slot of the differential.

**Attention!**

The allen bolt (1) must be tightened with a torque of 9 Nm (6.6 ft-lbs). If the bolt is tightened with more torque, the bearing could be damaged.

11. Mount the wheels and adjust the brake calipers.

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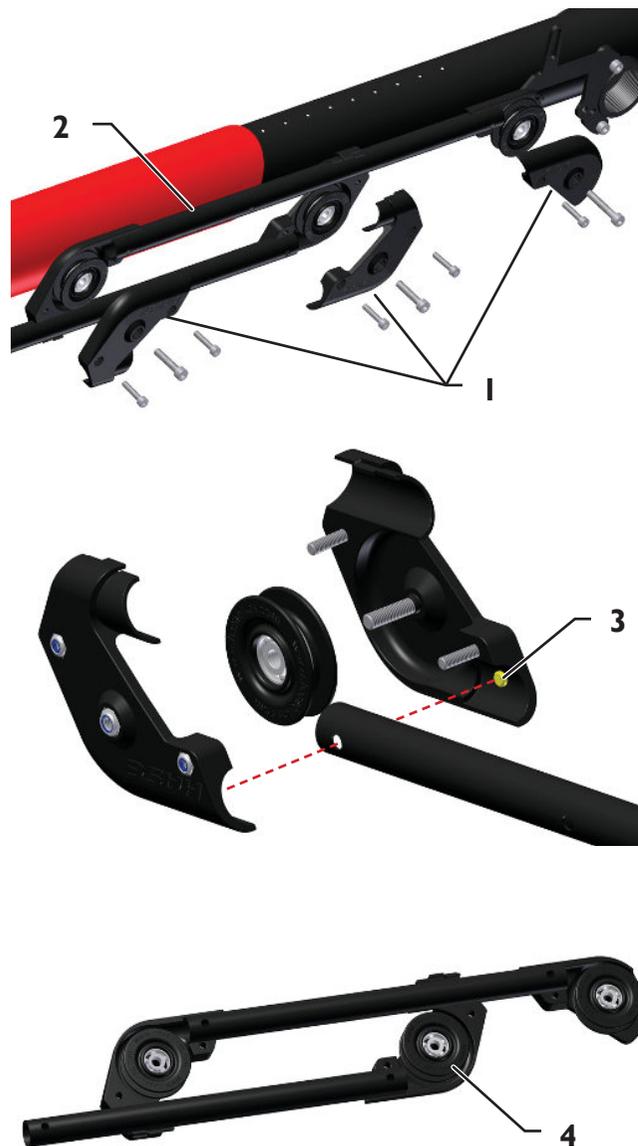
### Adjusting the quick-adjust assembly

In order to adjust the Trix to lengths outside of the range of the quick-adjust assembly (e.g. for larger riders), changes must be made to the quick-adjust assembly. After the quick-adjust assembly has been modified, the Trix will have a different range for quick length adjustments. This range can accommodate riders from approx. 1.65 to 2.00 m (5'5" to 6'7") in height.

1. Open the chain at the master link. Pull the chain out of the quick-adjust assembly.
2. Open the three idler pulley units<sup>(1)</sup> by removing the M<sup>4</sup> Allen bolts with a 3mm Allen key and the M<sup>5</sup> Allen bolts with a 4mm Allen key.
3. Remove the whole quick-adjust assembly from the frame. Pull the aluminum tubes<sup>(2)</sup> out of the idler pulley units.

#### Note:

The ends of the tubes fit tightly in the housing of the idler pulley units. Be careful not to damage the small protrusions (3) in the housing of the idler pulley units. The holes in the tubes must be aligned with the protrusions in the idler pulley housing when reassembling the quick-release mechanism.

7. Take the longer tubes from the retrofit kit and then reassemble the modified quick-adjust mechanism, referring to the lower middle diagram on the left or the exploded view of assembly on page 36. Begin by laying out the left (frame-side) halves of the idler pulley housing and connecting them with the aluminum tubes, taking special care to line up the holes in the tubes with the small protrusions in the housing. Then place the idler pulleys<sup>(4)</sup> in the housing, ensuring that the inner ring of each ball bearing is properly seated on the collar in the center of the respective bearing seat.
8. Place the right (outer) halves of the idler pulley housing onto the left halves and fasten them using the M<sup>4</sup> Allen bolts (3mm Allen key) and the M<sup>5</sup> Allen bolts (4mm Allen key).
9. Insert the end of Teflon tube (protruding from the lower aluminum tube) through the compression spring in the sleeve of the frame clamp. The aluminum tube will be pressing against this spring. Mount the small front idler pulley unit to the chain tube mounting bracket on the front boom using the M<sup>5</sup> Allen bolt.

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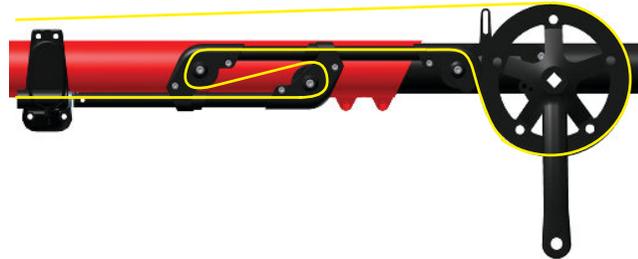
10. Lengthen the chain by 9 chain links (228 mm / 9").
11. Feed the chain through the quick-adjust assembly as shown in the bottom diagram on the left and connect the ends of the chain using the master link.

**Note:**

The process of feeding the chain through the quick-adjust assembly is easier when the Trix is resting on its rear wheels and seat (with the front wheel in the air).

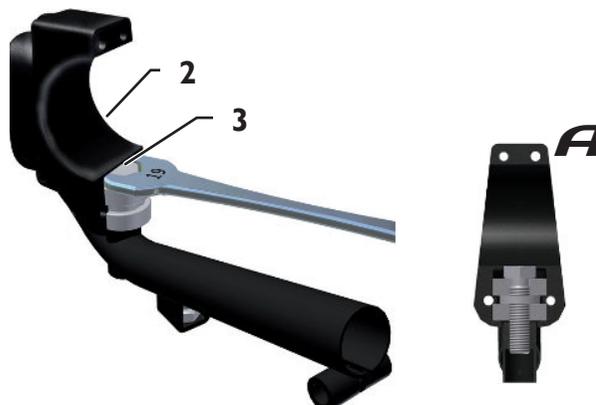
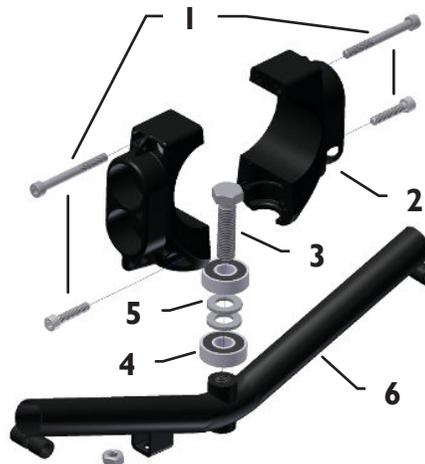
**Note:**

Move the frame clamp to the front if the shortest setting after reconstruction is too large (for riders whose height is about 1.65 m (5'5")).

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### Replacing the handlebar bearing

1. Unfasten the seat cover behind and beneath the seat: Push the two seat tubes apart. Pull them out of the sockets.
2. Open the frame clamp by removing the four M<sup>6</sup> Allen bolts (1) and separating the two clamp halves (2).
3. Unscrew the M<sup>12</sup> hex bolt (3) from the handlebar (6) using a 19 mm wrench (spanner).
4. Remove the two deep groove ball bearings.
5. To align the new deep groove ball bearings, place them with the washers and the M<sup>12</sup> bolt in one of the frame-clamp halves, as shown in diagram A.
6. Apply high-strength threadlocker to the lower end of the M<sup>12</sup> bolt (3).
7. Brace the M<sup>12</sup> bolt by holding it with a 19 mm wrench (spanner) and screw on the handlebar, tightening lightly.
8. Now remove the frame-clamp half (2) and tighten the M<sup>12</sup> bolt (3) to the handlebar (6) with a torque of 80 Nm (59 ft-lbs).
9. Rejoin the two halves of the frame clamp (2), encasing both the frame and the bolted bearings. Screw the frame-clamp halves together with the four M<sup>6</sup> Allen bolts, tightening lightly.
10. Adjust the position of the frame clamp so that it is level and situated 180 mm (7") from the end of the frame tube.
11. Tighten first the two lower Allen bolts (1) with a torque of 9 Nm (6.6 ft-lbs) and then the two upper Allen bolts (2) with a torque of 9 Nm (6.6 ft-lbs).
12. Insert the two seat tubes in the sockets of the frame clamp and refasten the seat cover.

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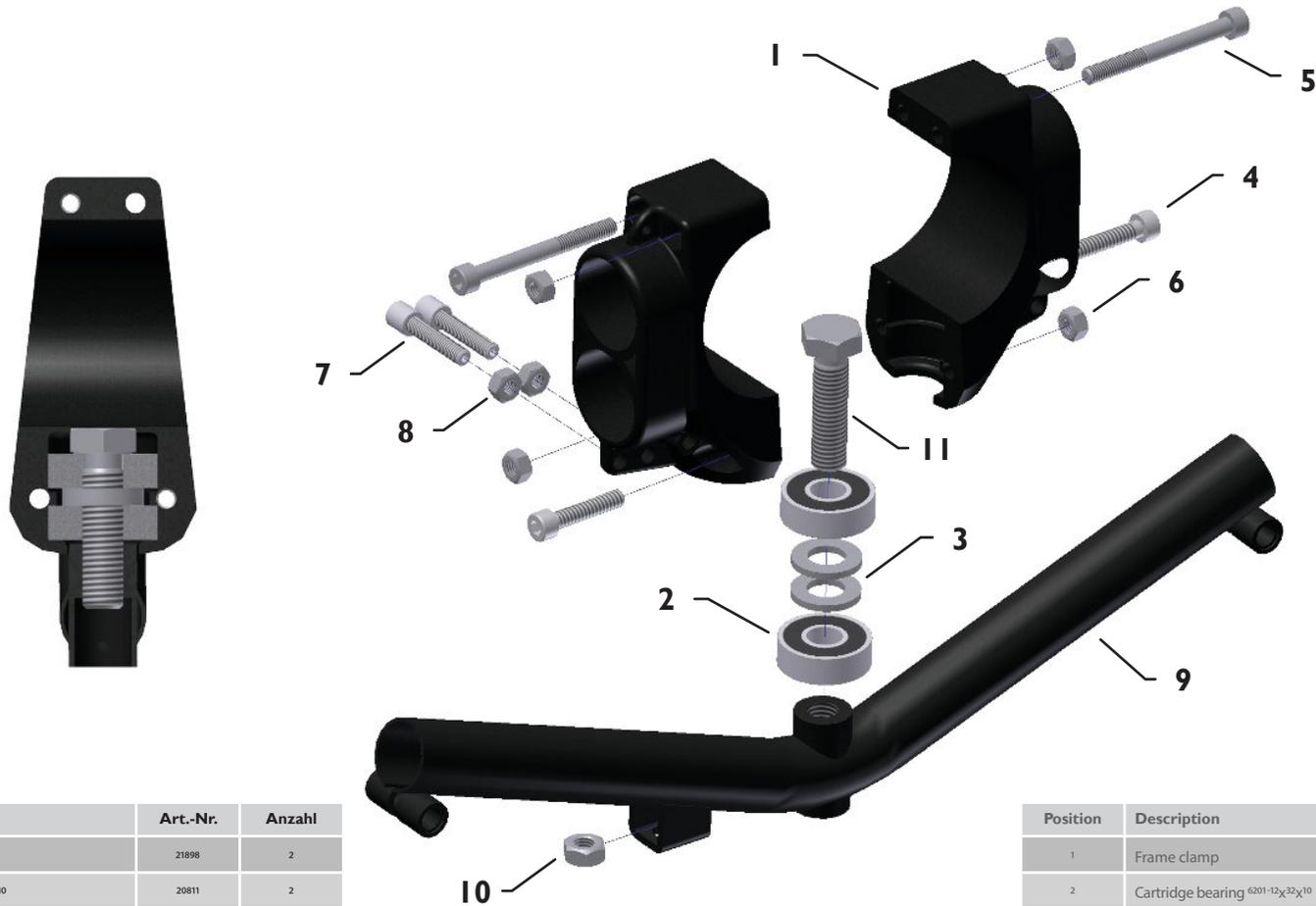
Position	Description	Art. No.	Number
1	3D seat cover	24917	1
2	Handlebar grips	21369	2
3	Headlight	21652	1
4	16" fork	24894	1
5	16" front wheel Trix	24123	1
6	Steering rod with stop	22758	1
7	Frame quickadjust assembly	25028	1
8	Frame clamp	21898	2
9	Parking brake mechanical	22332	1
10	Fenders 20", rear normal/wide	24289/24290	2
11	20" rear wheel normal/wide	24128/23848	2
12	Taillight	21653	1
13	Seat stay	24878	1
14	Differential	22750	1
15	Pedals with Toe clips and straps	22442/22445	1/2

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EN

## Terms of limited warranty

The terms and conditions are based on the statutory warranty. This excludes parts subject to wear, such as the chain, tires, brakes, etc. The warranty period (statutory) of <sup>24</sup> months commences on the date of purchase. In addition, Hase Bikes offers a <sup>3</sup>-year warranty (from date of purchase) against breakage of the frame in accordance with the following conditions. By sending in the enclosed registration card, you can extend the warranty period to <sup>5</sup> years. Your dealer is obliged to ensure, among other things, that your bicycle has no defects that diminish its value or suitability for the intended use. The exact details will vary according to your country. In Germany, this liability ends two years after purchase. Your dealer must assemble and adjust your bicycle to ensure safe function. The dealer also has to perform a final safety check and a test ride.

- Hase Bikes will only be held liable for defects resulting from faulty frame manufacturing and assembly. For assembly parts, the relevant manufacturer is liable.
- This additional warranty only applies to the original owner and to usage in accordance with the user's manual.
- Within the warranty period of <sup>3</sup> years, Hase Bikes will repair, or if necessary replace, any broken frame part free of charge. Any defective parts replaced by Hase Bikes become the property of Hase Bikes.
- Any further claims beyond the scope of this warranty are excluded. Hase Bikes will not cover the costs of assembly or disassembly (e.g. by the retailer) or shipping. We will not accept postal deliveries with insufficient postage.
- The maximum permissible load (rider and luggage) of <sup>140</sup>kg (<sup>308</sup> lbs) specified for the Trix must not be exceeded.
- If the Trix is used in competition, ridden on rough terrain, or subjected to abnormally excessive stress, the statutory and additional warranty becomes void.
- Damage caused by accidents is not covered by the warranty.
- The warranty becomes void if the Trix has been fitted with incompatible accessories or if accessories are installed incorrectly. Therefore always have components replaced and installed by an authorized retailer.
- The warranty becomes void if improper modifications or alterations (e.g. grinding, drilling, bending, etc.) are made to the frame.
- It is not possible to extend the warranty or commence a new warranty period.
- Warranty claims can only be handled by the authorized Hase Bikes retailer from whom the trike was purchased or by one of our distribution partners.
- The Trix may only be ridden by children under parental supervision.

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Have your vehicle inspected by your bicycle dealer at regular intervals. During the high season 'March through September', it is best to schedule any required repairs or inspections in advance in order to avoid unnecessarily long downtimes and waiting periods.

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### Checklist:

- Check pedals for tightness
- Check headset
- Check handlebar for tightness
- Check front and rear brakes
- Check front and rear wheel bearing
- Check bottom bracket
- Check cranks for tightness
- Check seat for tightness
- Check chain tension and abrasion
- Check wheels and air pressure
- Check rims and spokes
- Check all screws for tightness
- Check gearshift
- Check lighting system
- Check accessoires like motor system

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